

Glove Donning Guide

Do it right first time....

This is your step by step guide on how to put on a pair of single use gloves.



Step 1



Remove the Bastion single use Glove by the cuff from the dispenser

Step 2



Hold the wrist end of the glove open using your thumb and four fingers

Step 3



Ease your fingers then palm and thumb of the other hand inside the glove

Step 4



Gently pull the wrist end of the glove with your thumb and fingers while easing the hand fully into the glove

Step 5



Pull the glove to a comfortable fit then begin the procedure again for your other hand.

TAKE NOTE:

Donning a glove incorrectly can often cause gloves to rip and puncture, this is generally not due to a faulty glove.

Email: sales@gloves4u.eu

Tel: 0113 237 2555

Web: www.gloves4u.eu